

Rainbow Medical Devices Viscera Retainer

R_x ONLY

STERILE R
STERILIZED USING
IRRADIATION



WARNINGS/PRECAUTIONS

Prior to Use

- Inspect packaging before opening. The contents of the package are sterile if the package has not been opened or damaged. Do not use the device if the sterility has been compromised.
- Prior to use, all devices should be inspected to insure proper function and condition.
- Never use a damaged device.
- Confirm that the retainer is properly located before suturing.
- Protect patient tissue at all times during use.

During Use

- Ensure device is used by a trained and licensed practitioner.
- Avoid excessive force during use.

After Use





- Viscera Retainers are intended for single use and for single patient use. Do not reuse.
- Dispose of used Viscera Retainers according to internal hospital guidelines.

CONTRAINDICATIONS

Single use only. Do not reprocess. Risks associated with reprocessing/reuse: This device has not been validated for reprocessing/reuse. The design of the device may not facilitate cleaning and if inadequately decontaminated may result in risk of infection. The device may not perform as intended by the manufacturer if it is reprocessed and/or decontaminated and may result in material degradation resulting in failure of the device.

INSTRUCTIONS FOR USE

The Viscera Retainer is designed and made to retain and protect the omentum and viscera during closure of the peritoneal cavity. It provides added anchorage and retentive power to speed abdominal closure and significantly reduce the risk of nicks and punctures.

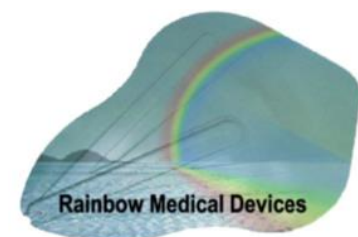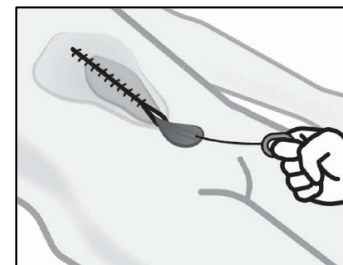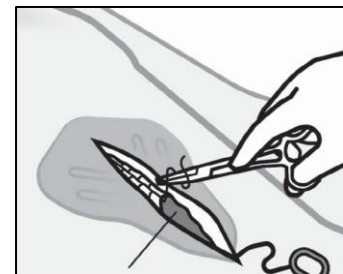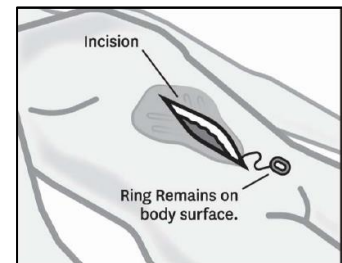

Manufactured for:
Rainbow Medical Devices
Gurnee, IL

Made in China
IFU-001-2015